SAESHIN

traus B110/B111

INSTRUCTION MANUAL

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Contents

- Introduction
- Precaution
- Features & Benefits
- Components
- Component Names
- Control Panel Diagram/Buttons
- Set up of Control Box
- Operation Procedure
- Maintenance
- Specification
- Warranty
- Symbols

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Introduction

Please read this manual regarding the instructions for handling method, maintenance and inspection prior to use and store it in a place where users can read it anytime.

This manual is for any users to install and operate appropriately in order to have the long term usage with the equipment.

Please pay a special attention to the  ,  and  signs in order to prevent any risks.

<Classification of equipment>

- Type of protection against electric shock
 - Control Box: Class II appliance
 - Handpiece: Class III construction
 - Foot Controller: Class III construction
- Not suitable with any flammable gas containing oxygen or gas anesthetic.
- Types of the operation mode: Products with any occasional overloading operations.

**Product design and specification can be changed without any notice
for the quality improvement.**

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Precaution

Caution

- The product has been produced with two power source for user's convenience. If the product doesn't turn on, make sure both 'Power Switch' on the back and 'Controller ON/OFF Button' on the side are both on. (Refer to 'Component Names')
- If the product is not being used for a long time (12+ hours), please make sure to turn the Power Switch off on the back as well. It could be a reason on fire or accident.
- Do not operate it as the Handpiece's Collet-chuck is open. It can cause damage to the Handpiece.
- Do not open the Collet-chuck as the Handpiece is operating.
- Check if the Power Switch is off before you connect the Power Cord to minimize the danger of a sudden operation.
- A circuit breaker is installed for the motor protection. The breaker will work when there is overheat that the motor can't handle. Please operate in the range that the breaker will not be in use.
- Please store the product as the Bur is attached to the Collet-chuck.
- Change the rotate direction when the Handpiece is stopped.
- Do not use the modified Bur. It could cause a breakdown or an accident.
- Do not use in dangerous places or humid places.
- Always wear appropriate outfit and protection gears. You must wear the protection goggle with a mask, and use an appropriate inspirator and a ventilator.
- Do not dissemble or amend the product on your own. It could be a reason of an accident.
- Please only use appropriate Handpiece and Foot Controller for the main product.
- Please check the problems such as excessive vibration of a Bur, and malfunction of speed control.
- Please gently control the Speed Control Dial.
- Please change the Bur when the Handpiece is stopped.
- Be careful to not drop the product. If the product is dropped, make sure it does not produce excessive heat and or if the Bur is bent, as you operate the product very gently.
- If the product causes troubles, please use it after getting it fixed.
- Do not put the Handpiece down until it is completely stopped.
- Use appropriate voltage.
- Use appropriate accessories.
- Follow the operation manual for appropriate speed for multiple purposes.
- Equip appropriate production equipments (dirt mask, protection goggle, tec.)
- Keep the user environment clean. Dirt can cause accidents.
- Do not touch the Power Cord or Handpiece Connector with wet hands.
- Do not over bent the Power Cord or press it with the heavy object. It can cause a fire or an electric shock.
- Do not place the product near heater, candle, lighter, etc. It can cause a fire.
- Do not use excessive number of electronics on the multi-tap. It can cause a fire or an electric shock.
- "This appliance can be used by children aged from 8 years and above and person with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in safety way and understand the hazards involved."
"Children shall not play with the appliance."
"Cleaning and user maintenance shall not be made by children without supervision."
- "If the supply cord is damaged, it must be replaced by the manufacturer, its service agent of similarly qualified persons in order to avoid hazard."

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Features & Benefits

■ B110

- Convenient Power Switch
- Knee/Foot Type Pedal
- Sensor Type BLDC Hand Piece with 'H180' applied
- Maximum Rotation 50,000 rpm (Options to change the maximum RPM / 40,000 rpm <-> 50,000 rpm)
- Maximum Torque 7.0 N-cm
- Forward/Reverse Selector Switch (F/R)
- Standard Bur $\Phi 2.35\text{mm}$
- Method for connecting any Collet-chuck type Bur
- Auto-cruise Function
- Prevention for Overloading

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Components

■ B110

· Please check the list for the components

No.	Components
1	Control Box (B110)
2	Handpiece (H180)
3	Foot Controller
4	Power Cord
5	Handpiece Stand
6	Wrench (2 Types)
7	Adhesive Wall Mount Bracket
8	Screws, Bolts, Nuts

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Components

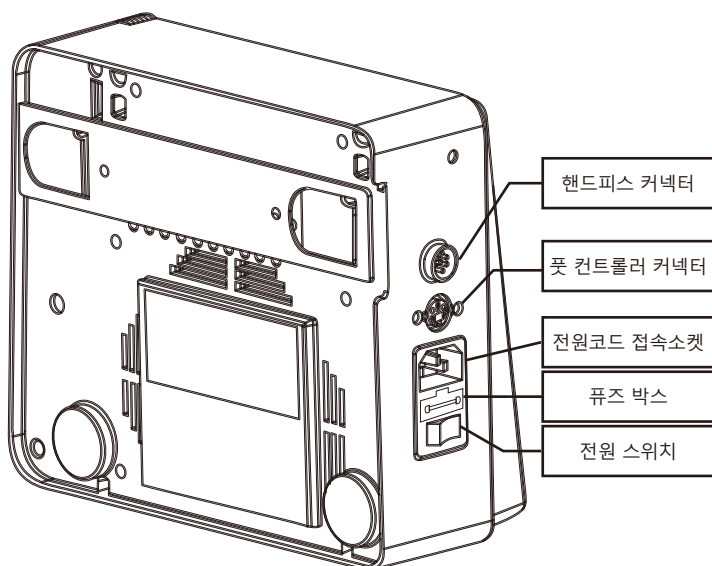
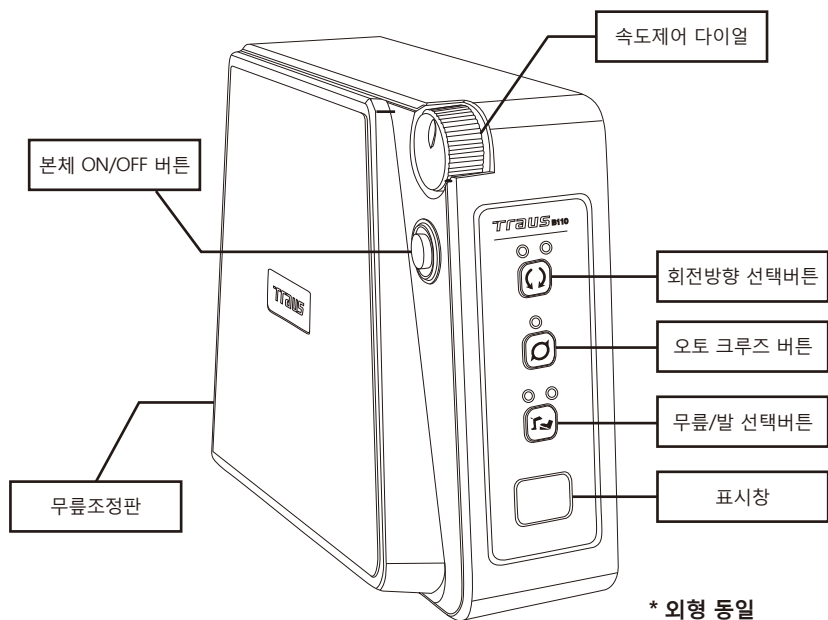
■ B111

· Please check the list for the components



No.	Components
1	Control Box (B111)
2	Handpiece (H160)
3	Foot Controller
4	Power Cord
5	Handpiece Stand
6	Adhesive Wall Mount Bracket
7	Screws, Bolts, Nuts

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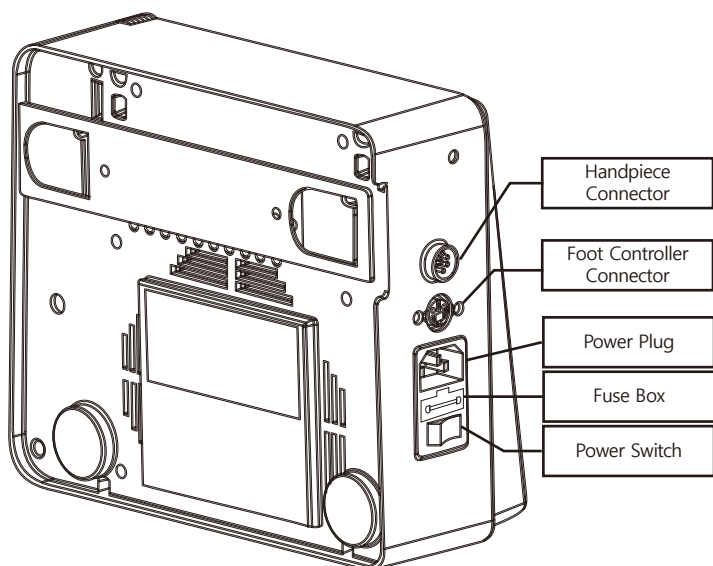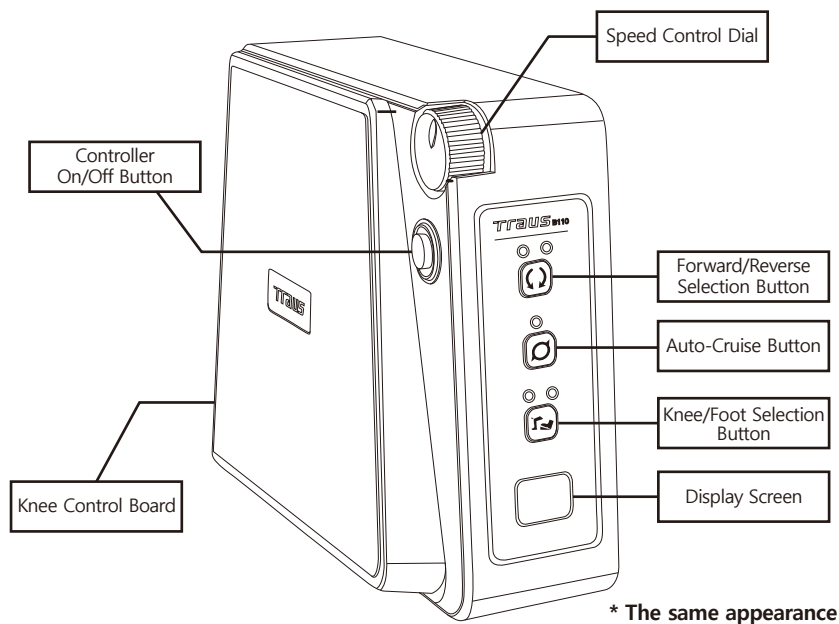■ Speed Control Dial:

Use this dial to control the speed.

Speed increases when rotated counter clockwise, and decrease clockwise.

• B110 How to set Max RPM

A max of 40,000 rpm can be set up when rotate the counter clockwise from the default. If you press the Speed Control Dial for about a second, you will hear two beeping sound, and you can go up to 50,000 rpm if you rotate more counter clockwise. When it's set up to 50,000 rpm, if you press the dial again, you will hear one beeping sound and the product will be back to 40,000rpm.

■ Forward/Reverse Selection Button:

A button that can select the rotate direction to counter clockwise or clockwise.

■ Auto Cruise Button:

A button that keeps the speed of the Handpiece fixed.

Please operate the Handpiece in the speed you want as the auto cruise feature is activated. If you keep it for about 3-5 seconds, the speed will be fixed with a beeping sound. If you want to deactivate this feature, please turn off the Auto-Cruise Button or press the Foot Controller once.

■ Knee/Foot Selection Button:

The button that makes you choose to operate the Handpiece by knee or foot.

- Knee: Operate the Handpiece by pressing Knee Control Board Button.
- Foot: Operate the Handpiece by pressing the Foot Controller.

■ Display Screen:

- Rotate count: Display RPM. Actual rotation count is displayed number x 1,000.
- Error code: Display error when it is detected.

■ Power Switch(Rear):

Use to turn the product on/off

- | : Power is on.
- O : Power is off

■ Controller ON/OFF Button(Side):

Used to turn the product on/off when the Power Switch(Rear) is on.

Press this button once to turn the lamp and the product on. Press once more to turn the lamp and the product off.

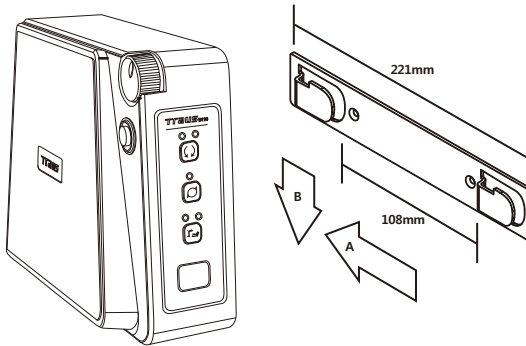
- Only available when the Power Switch is off.

■ Knee Controller:

The button that operate the Handpiece when you select Knee/Foot Selection Button to knee.

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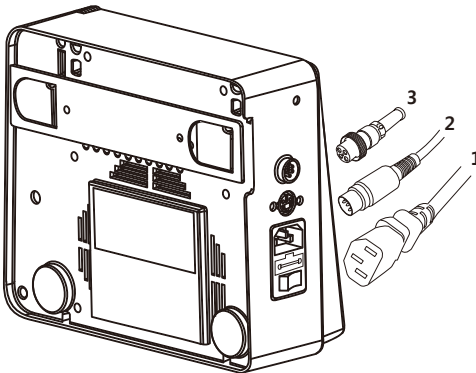
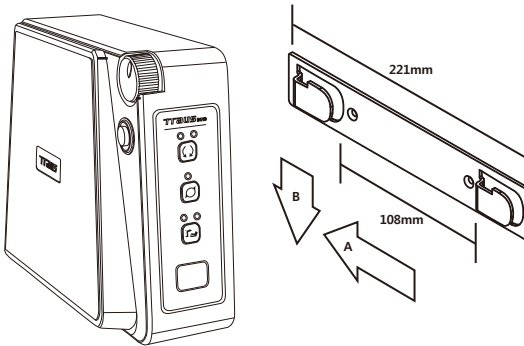
Set Up of Control Box

■ Installation for Adhesive Wall Mount Bracket


1. Refer to the image and attach the adhesive Wall Mount Bracket to the bottom surface within the user's knee height by the Bolts provided.

- Please do not let the bolt head stick out from the Bracket's surface.

2. Make sure the hole is placed correctly and attach the Control Box to the Bracket by the direction from A -> B as shown from the image above.

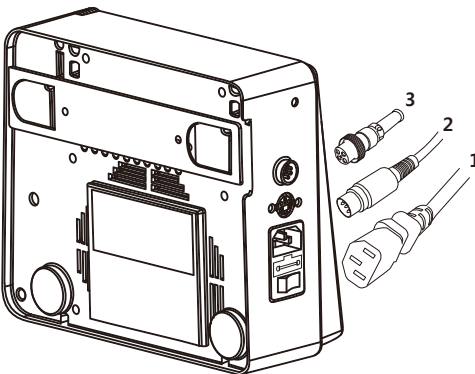■ Installation for Control Box

1. Plug in the Power Cord to the Power Plug.

- Make sure that the Power Switch is off before connecting the Power Cord.

2. Connect the Foot Controller Cord.
3. Connect the Handpiece Cord.

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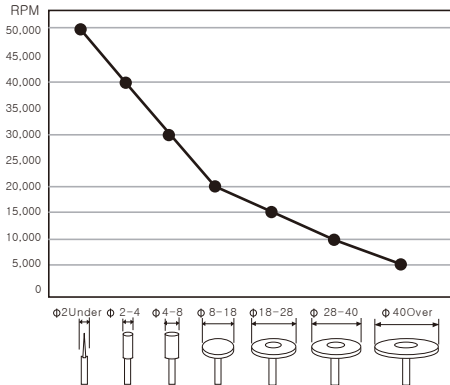
■ Bur Set Up

1. Rotate the Handpiece handle and open the chuck.

2. Insert the Bur into the Handpiece.

3. Rotate the Handpiece handle and close the chuck.

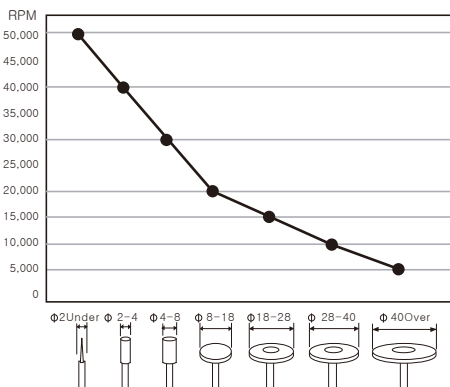
※ Connecting Handpiece Bur is same regardless of any Handpiece types.

- Close the chuck before using the Handpiece. It may cause the damage.
- Check if the Bur is not damaged.
- Check the size of Bur. (Bur dimension: $\Phi 2.35\text{mm}$)

※ The chart below shows the eligible rpm per Bur size.

(Speed may be different due to the Bur's materials and types.)

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Operation Procedure

1. Grab the Handpiece with one hand and use the other hand to turn the Power Switch(Rear) on.
(When the Power Switch is on, the green lamp on the switch will lit.)
2. Turn the product on by pressing Controller ON/OFF Button(Side).
(When the product is on, the green lamp on the button will be lit with a beeping sound, and the RPM will be displayed on the screen.)
3. Choose the rotation direction by pressing Forward/Reverse Selection Button. When the left light is on, it's counter clockwise, and when the right lamp is on, it's clockwise.
4. Choose the control method by pressing Knee/Foot Selection Button. When the left light is on, knee is selected, and when the right lamp is on, foot is selected.

■ How to operate when knee is selected.

5. Use Speed Control Dial to choose ideal speed.
6. Operate Handpiece as you control the Knee Control Board. (It will operate within the range.)
7. When you unattach the Knee Control Board, the Handpiece stops.

■ How to operate when foot is selected.

5. Use Speed Control Dial to choose ideal speed.
6. Operate Handpiece as you control the Foot Controller. (It will operate within the range.)
7. When you unattach the Foot Controller, the Handpiece stops.

■ Auto Cruise Feature

A feature that keeps the speed of the Handpiece fixed.

Operate the Handpiece in your ideal speed as you have the auto cruise feature activated. If you keep it for 3-5 seconds, the speed will be fixed with a beeping sound. If you want to deactivate, press the button one more time or press the Knee Control Board or Foot Controller once.

■ Last Memory Feature

A feature that stores the last condition when the power turns off.

Control Box saves the set up (Knee/Foot, Rotation direction, Auto-Cruise selection) at the time it was turned off.

- Check the set up when you turn it on. It cause an accident.
- B110 saves RPM additionally at the time you operated the Handpiece finally.

■ Overheat Prevention Feature

When the Handpiece is overheated, it will be automatically turned off after about 5 seconds when the error code E1 is displayed on the screen with a beeping sound.

If you wish to deactivate, press the Knee Control Board or Foot Controller once.

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■ Error Code

- B110

Error Code	Cause	Action
E0	<ul style="list-style-type: none"> · Faulty connection of Handpiece · Abnormal operation of Motor 	<ul style="list-style-type: none"> · Check if the power cord or the Handpiece cord is connected correctly. · Turn the power on again. · If is not solved, please contact the manufacturer.
	<ul style="list-style-type: none"> · Handpiece overheat 	<ul style="list-style-type: none"> · Press the Knee Control Board or Foot Controller once. · Please cool the product down for about 10 minutes at cold place before re-operation. · If is not solved, please contact the manufacturer.
	<ul style="list-style-type: none"> · Operate with open chuck 	<ul style="list-style-type: none"> · Please operate after closing the Handpiece's chuck.
E1	<ul style="list-style-type: none"> · Faulty Handpiece assembly · Faulty Bearing · Faulty Motor 	<ul style="list-style-type: none"> · Please contact the manufacturer

- B111

Error Code	Cause	Action
E1	<ul style="list-style-type: none"> · Handpiece overheat 	<ul style="list-style-type: none"> · Press the Knee Control Board or Foot Controller once. · Please cool the product down for about 10 minutes at cold place before re-operation. · If is not solved, please contact the manufacturer.
	<ul style="list-style-type: none"> · Operate with open chuck 	<ul style="list-style-type: none"> · Please operate after closing the Handpiece's chuck.
	<ul style="list-style-type: none"> · Faulty Handpiece assembly · Faulty Bearing · Faulty Motor 	<ul style="list-style-type: none"> · Please contact the manufacturer

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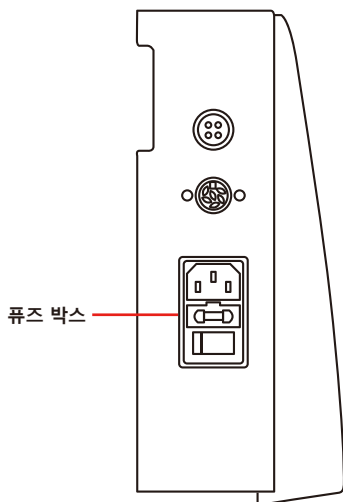
- AC 100-120 V: 5 A 퓨즈
- AC 220-240 V: 2 A 퓨즈

Maintenance

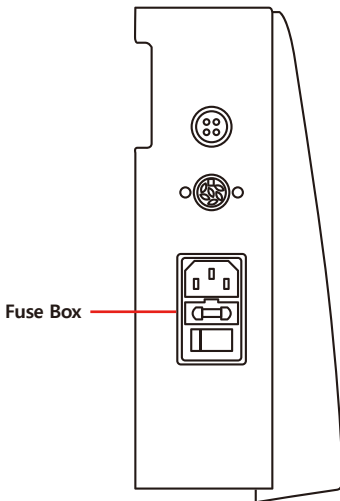
1. Please keep the area of Handpiece clean.
2. Remove the dust with alcohol and avoid any humid conditions for the Handpiece.

- Do not use oil on the product. Do not clean with water.
- Lubricating the inside of Bearing with oil may damage the product.

3. Air cleaning is possible but it may affect the Handpiece or Motor with the air pressure.
4. Store the product with the bur placed into the collet chuck.

■ Fuse Exchange


Take out the Fuse Box to exchange the Fuse and place back to where it was before.

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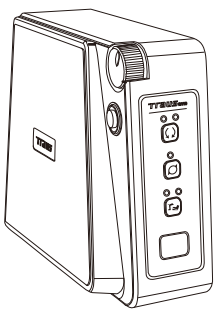Warranty

Saeshin covers any defects in product or workmanship under normal use during the warranty period (1 year). During the Warranty Period, Saeshin will repair or replace at no charge, products or parts of a product that proves defective because of improper material or workmanship, under normal use and maintenance. However the Warranty excludes any products caused by misuse of product, incorrect installation, inappropriate maintenance and repair from users by using improper bearing, spindle, gear and other parts.

Warranty Exclusions within the Warranty Period

- Failure to comply with the manufacturer's written instructions
- Failure to correctly adjust electricity voltage to the manufacturers stated requirements
- Damage whether accidental, willful or intentional
- Use of unauthorized parts or accessories
- Use of unauthorized service, repairs and alterations
- Normal wear and tear (Ball bearing, Gear, Chuck, Spindle case Assy, Armature etc)
- Act of God

Conditions for Operations	Temperature (°C)	10~30
	Humidity (%)	10~80
	Atmospheric Pressure (hPa)	700~1,060
Shipment / Storage Conditions	Temperature (°C)	-20~60
	Humidity (%)	10~90
	Atmospheric Pressure (hPa)	500~1,060

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Symbols

	ON (Power Switch)		OFF (Power Switch)
	Manufacturer		Refer to Instruction
	EU Representative		General Warning Sign
	Serial Number		Class II Appliance
	Fuse		Alternating Current
	Do not dispose with standard industrial waste materials.		CE Marking

MEMO

A large, empty rectangular area with rounded corners, intended for writing a memo.

MEMO

A large, empty rectangular area with rounded corners, intended for writing a memo. The area is light gray and occupies most of the page below the header.

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SAESHIN PRECISION CO., LTD.