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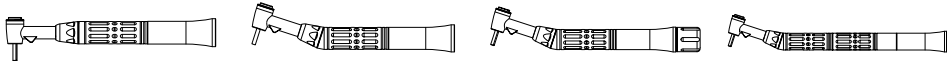
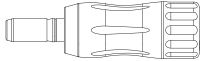
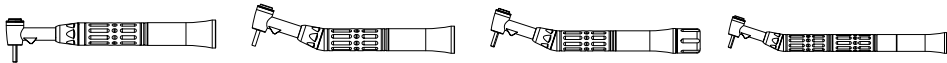
**EC** **REP**  
MERIDIUS MEDICAL EUROPE LIMITED  
Unit 3D, North Point House North point  
Business Park New Mallow Road CORK,  
T23AT2P, Ireland  
+353 212066448


SOM-S054S-K [Rev.5] 230201

Before the operation, read the instruction manual carefully and keep it for future reference.

Angle Handpiece

Model	TSH11XX	CSH11XX	CSH20NN	CSH30NN
Feature	Straight Angle	Contra Angle	Contra Angle	Contra Angle
Weight	50g	50g	61g	43g

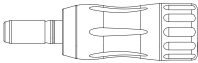Torque Converter

Model	TC00	TC10	TC20	TC35
Torque[Ncm]	No-limit	10	20	35
Color	Black	Red	Blue	Yellow
Weight	50g	50g	50g	50g

※ CSH20NN is All-in-one type which doesn't need proper Torque Converter.  
Rear handle will performs the same function and role as TC00.

1. Intended purpose

: Torque wrench used to precisely apply a specific torque to a fastener bolt for fixation of an abutment, dentures or prosthetics on a dental implant.

2. Components

Angle Handpiece : Angle, Spanner  
Torque Converter : Torque Converter, Handle Ring(Optional)

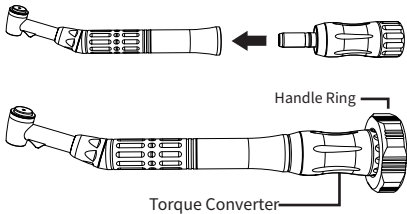
3. Product feature & Advantages

- 1) It is suitable for procedures requiring precise torque.
- 2) More precise, faster, and safer procedures are available.
- 3) Torque Converter and Angle Handpiece are easy to connection and separation.
- 4) Angle Handpiece can be sterilized.
- 5) There are four types of Torque Converter, No-limit, 10 N-cm, 20 N-cm, and 35 N-cm, and the choices are available according to the needs of the user.
- 6) Angle Handpiece's head form is a button type.

4. Connection of each part

1) Connection the Angle Handpiece and Torque Converter  
: Put the Handle Ring into Torque Converter, after inserting proper Torque Converter into sterilized Angle Handpiece

2) Separation the Angle Handpiece and Torque Converter  
: Disconnect Handle Ring from Torque Converter.  
After, hold the Angle Handpiece and Torque Converter with both hands and pull to separate.

※ The Handle Ring is optional.

5. How to Use

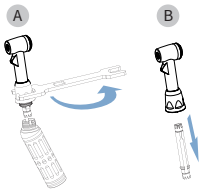
- 1) Insert a Torque Driver tip of same form of implant fixture and abutment.
- 2) Make sure the Torque Driver tip is correctly secured after connecting.
- 3) While slowly rotating the Handle Ring to the left or right with your finger, make progress implant surgery by tighten or loosen the screw.

6. Maintenance of the product


1) Cleaning  
: Separation the Angle Handpiece and Torque Converter, then wipe the dust removal and surfaces with a cotton swap or paper towel.

2) Lubrication

- ① Disassemble the product into 3 parts.
  - A. Fixed a Wrench to the joint cap and turn joint cap counterclockwise to open.
  - B. Take out the gear.

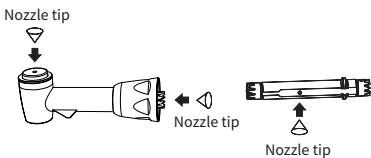② See the picture below, put the nozzle with the right direction and spray lubricant for 2~3 seconds.

③ Reassemble the product.

- A. Insert the gear into the head assembly.
- B. Make sure the gear fits into the head assembly and check it turning smoothly.
- C. Assemble the head part and body as right direction.
- D. Use a Wrench to lock the joint cap tightly.

- The nozzle tip is not provided by the manufacturer.
- After lubrication, please put the gear in the right direction. Gear in the wrong direction can damage the angle handpiece.
- If working with the Torque Driver for a long time, please lubricate it every hour.
- When lubricating the Torque Driver, please hold the Angle Handpiece firmly. The Angle Handpiece could be soaring up by gas pressure of the oil sprayer.

3) Sterilization

- ① Put the angle handpiece in a sterilization pouch and seal up the pouch.
- ② Place the pouch in the autoclave machine. Sterilize for 4 minutes at 132°C and dry for 30 minutes.

Sterilization	Temperature	Time	Dry
Steam sterilization	132°C	4 minutes	30 minutes

- The maximum number of sterilizations is 250 times.
- The steam sterilization is recommended. Do not have the other sterilizations.
- Please put the product in a sterilization pouch and seal up the pouch before sterilization.
- Please dry the product completely before sterilization.
- Please clean and lubricate the product before sterilization. Otherwise, contaminants inside the product could lead to a breakdown.
- Please dry the product completely after sterilization.
- Do not sterilize the Torque Converter.

[Caution for use]

- This product can be used for intended purpose only.
- Follow all instructions in manual.
- Always be aware of the safety and check the product before using it every time.
- Do not change and disassemble, except mentioning in manual.
- Do not drop product on floor and do not give any damage.
- If the product is operating abnormally, stop immediately and ask your seller for a product check.
- Please make sure that the top priority is the safety of the patient when using the Torque Driver. This requires sufficient caution when used.
- This product is not in sterilization once release by manufacturer. Please certainly do user sterilization before using. (Refer to 'Maintenance of product - sterilization')
- Please make sure to clean the blood inside of the product before sterilization. Unless, the product can be damaged.
- When you sterilize, please follow the manual.
- When using Angle Handpiece, tissue inside the patient's mouth should always be protected.
- Do not force to insert Torque Converter on Angle handpiece.
- Please lubricate both before and after use.
- Please contact the dealer/store or manufacturer when problem occurs.
- Use it differently depending on the patient's oral health and use.
- Incorrect use may damage the product or nearby bone tissue. Be sure to use it by an experienced technician or specialist.
- Maintenance is essential to maintain durability and performance of the product.

Operating condition	Transport and storage condition
Temperature : 10~35°C	Temperature : -20~60°C
Humidity : 10~80%	Humidity : 10~90%
Atmospheric pressure : 700~1,060hPa	Atmospheric pressure : 500~1,060hPa