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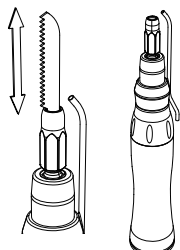
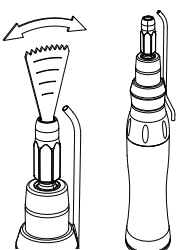
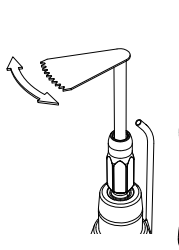
모델명	TRAUS TRT12NRC	TRAUS TRT12NSG	TRAUS TRT12NOS
전 용 블 레이 드	RECIPROCATING BLADES	SAGITTAL BLADES	OSCILLATING BLADES
	① TRAUS SAB-R15  15mm - Blade Thickness 0.4mm	① TRAUS SAB-S6  6mm 25mm - Blade Thickness 0.4mm	① TRAUS SAB-O8  8mm EO - Blade Thickness 0.4mm
	② TRAUS SAB-R20  20mm - Blade Thickness 0.4mm	② TRAUS SAB-S10  10mm 25mm - Blade Thickness 0.4mm	② TRAUS SAB-O15  15mm - Blade Thickness 0.4mm
	③ TRAUS SAB-R30  30mm - Blade Thickness 0.4mm	- Blade Thickness 0.4mm	- Blade Thickness 0.4mm

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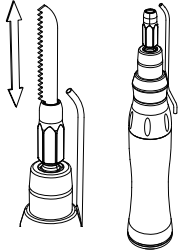
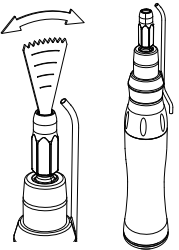
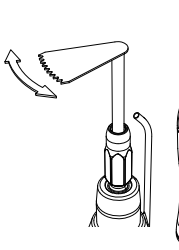
Traus

Micro Saw Handpiece

Please read the instruction manual carefully before operation and keep it for future reference.

Model	TRAUS TRT12NRC	TRAUS TRT12NSG	TRAUS TRT12NOS
Picture			

Weight	102g	102g	102g
Max. RPM	Max 12,500 rpm		
Irrigation Type	External Irrigation Type		
Blade Replacement Type	Locking-jaw Type		
Coupling Dimension	Standard Dimension of Dental Handpiece (ISO 3964)		

1. Intended Use

: For surgical bone cutting, shaving, polishing, grinding, drilling, endodontic treatment.

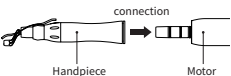
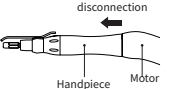
2. Components

: Handpiece, External Irrigation Nozzle, Spray Nozzle, Spanner, Instruction Manual

3. Connection & disconnection of handpiece and motor

- ① Put the handpiece directly into the micro motor in order to connect them.
- ② Hold the handpiece and the micro motor, and then pull the handpiece from the micro motor in order to disconnect them.

- 
- Please check if the connecting part is fastened tightly by pulling the handpiece from the micro motor smoothly.
 - Please do not connect or disconnect the handpiece until the micro motor has stopped completely.
 - Please use an E-type micro motor meeting ISO 3964 only.
 - Please connect or disconnect the handpiece not inserting a blade.

	
Handpiece and motor connection	Handpiece and motor disconnection

4. Blade Insertion & Removal

4.1 Insertion

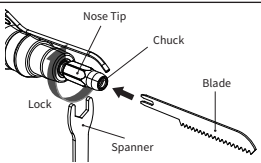
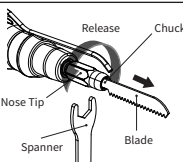
- ① Turn the spanner counterclockwise to release the nose tip.
- ② Insert the blade. Push the blade in order to insert completely.
- ③ Turn the spanner clockwise to tighten the nose tip.

- 
- Please check if the blade is fastened tightly before use.
 - Please use the Reciprocating blade, the Sagittal blade or the Oscillating blade provided by our company. (Option, refer to the article

4.2 Removal

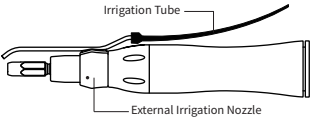
- ① Turn the spanner counterclockwise to release the nose tip.
- ② Remove the blade by pulling by hand.
- ③ After having removed the blade, turn the spanner clockwise, and tighten the nose tip lightly.

- 
- Please change the blade after the micro motor has stopped completely.
 - Please do not tighten or loosen excessively.
 - Do not use any tools other than the provided spanner.

	
Blade Insertion	Blade Removal

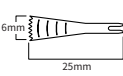
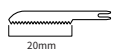
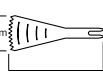
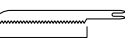
5. External Irrigation Nozzle Insertion & Tube Connection

- ① Connect the external irrigation nozzle on the handpiece.
- ② Insert the irrigation tube into the edge of the external irrigation tube. Please check if the tube is connected tightly.

6. Blade (Option)

- ① Mount the external irrigation nozzle on the handpiece.
- ② Insert the irrigation tube into the edge of the external irrigation nozzle. Please check if the tube is connected tightly.

Model	TRAUS TRT12NRC	TRAUS TRT12NSG	TRAUS TRT12NOS
Exclusive Blades	RECIPROCATING BLADES	SAGITTAL BLADES	OSCILLATING BLADES
	① TRAUS SAB-R15  15mm - Blade Thickness 0.4mm	① TRAUS SAB-S6  6mm 25mm - Blade Thickness 0.4mm	① TRAUS SAB-O8  8mm - Blade Thickness 0.4mm
	② TRAUS SAB-R20  20mm - Blade Thickness 0.4mm	② TRAUS SAB-S10  10mm 25mm - Blade Thickness 0.4mm	② TRAUS SAB-O15  15mm - Blade Thickness 0.4mm
	③ TRAUS SAB-R30  30mm - Blade Thickness 0.4mm		

7. Maintenance

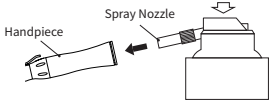
7.1 Cleaning

- ① Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
※ Do not put tap water in the motor.
- ② Disconnect the angle handpiece to the motor.
- ③ Remove contaminants on the angle handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- ④ Rinse the angle handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- ⑤ Wash the angle handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- ⑥ Rinse the angle handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- ⑦ Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.

- 
- Please make sure to clean, lubricate and autoclave the handpiece right after use before remaining blood inside the handpiece has been dried.
 - If remaining blood inside the handpiece has coagulated, the handpiece may get rusty.

7.2 Lubrication

- ① Insert the nozzle to motor assembly part(the end part of angle) and spray lubricant 2~3 seconds.

- 
- Lubricate the product after each use.
 - When lubrication, please hold the product firmly because the product can go out by the spray pressure.
 - If the continuous operation, please put the oil every an hour.

7.3 Sterilization

- ① Dry the product completely after cleaning.
- ② Put the product in a sterile bag and seal tightly.
- ③ Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132℃, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Dry Time	132℃ SSS
Moist heat sterilization	132℃	4 minutes	30 minutes	

- 
- This product is designed to withstand 250 times of re-sterilization process.
 - Make sure to use the sterilization bag and seal tightly.
 - Make sure to dry the applicable products before sterilization.
 - Sterilizing the product with blood or other remains may cause the breakage of the product. Therefore, it is necessary to clean the product before sterilization.
 - This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.

8. Warranty

SAESHIN guarantees this product for 1 year from invoice date. Product warranty means that SAESHIN has responsibility for defective material or operation failure. The product warranty does not cover user's misuse, wrong installation, inappropriate maintenance and repair, and normal wear of consumable parts such as ball bearing. In order for SAESHIN to confirm if the warranty is valid will require checking the operating condition, environment information, serial number and stamped invoice. The warranty will be performed as a repair or exchange according to manufacturer's analysis of product and judgement.

Warranty Exclusions

Misuse and improper handling by users
Dropping the products during installation, movement or use
If the unit is damaged due to unspecified consumable parts or accessories
If the product is not repaired by Saeshin or the repair shop designated by Saeshin.
Normal wear of consumable parts (Ball Bearing or etc.)
Damage and failure due to natural disaster
If the product is disassembled or modified at will.

- 
- [Precautions]

 - Use of any other purpose is prohibited.
 - The company did not sterilize the product. Please make sure to sterilize before operation. (Please refer to the 7.3 Sterilization)
 - Do not drop the product or force with strong external shocks.
 - Do not disassemble or modify the product.
 - Check for vibration, heat, noise and other abnormal symptoms before use.
 - If the product operates abnormally, stop immediately and contact the seller for check.
 - Do not remove the motor while the product is rotating.
 - Moisture and impurities may cause malfunction of the product. Please clean and dry with air.
 - Make patient safety the top priority when use.
 - To maintain it clean, always wipe the product with rubbing alcohol before use.
 - Improper use may cause damage to the product. Be sure to use it by a skilled technician or specialist.
 - The manufacturer is not responsible for accidents caused by use not in accordance with the instruction manual.
 - The user is responsible for the proper use and maintenance of the product.
 - This product should not be disposed with general waste.

Use conditions	Shipping/Storage conditions
Temperature : 10~35℃	Temperature : -20~60℃
Humidity : 10~80%	Humidity : 10~90%
Atmospheric pressure : 700~1,060hPa	Atmospheric pressure : 500~1,060hPa

This product is the medical device.

SAESHIN
SAESHIN PRECISION CO., LTD.
52, Secheon-ro 1-gil, Dasa-eup,
Dalseong-gun, Daegu, Korea

EC	REPF
MERIDIUS MEDICAL EUROPE LIMITED Unit 3D, North Point House North point Business Park New Mallow Road CORK, T23AT2P, Ireland +353 212066448	

Tel. 82-53-587-2341 / Fax. 82-53-580-0999
www.saeshin.com / traus@saeshin.com

SOM-S079P-KE [Rev.2] 2023-02-01