Digital Pencil Heater

# LabMuse

p.h

**SAESHIN**

[www.saeshin.com](http://www.saeshin.com) / [traus@saeshin.com](mailto:traus@saeshin.com)

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Thank you for your purchase.

In order to have trouble-free performance for many years, please read all instructions before operating.  
Please keep this manual for future reference.

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When you work for waxing up, this equipment is able to replace Bunsen burners, Spirit lamps, Alcohol or Gas burners. You can gain quickly high heat and it is safe and efficient to use.

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## CAUTIONS / 주의사항

- \* Do not touch the dipping unit with wet hands.
- \* The accurate circuit board is included in the dipping unit so that it is necessary to avoid any impact or force which may cause mechanical troubles.
- \* Contact the manufacturer or distributors for questions or more information regarding the trouble-shooting.

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**For your safety, Keep to the following / 안전을 위해 꼭 지켜주세요.**

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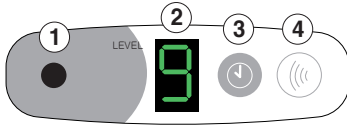
Saeshin is not responsible for the accident causing by not following the instructions.  
Design and specification of the product can be subject to be changed for improving the quality.  
The users there are operation and responsibility regarding maintenance.  
Informing the product, model, faulty status and the customer's contact information  
(tel # or email etc), the kind consulting and quick service will be provided.

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- |            |  |
|------------|--|
| ① 구분표시램프   | 1. lamp to show on operating                             |
| ② 단계표시 LCD | 2. LCD showing the level of sensitivity and heating time |
| ③ 시간설정버튼   | 3. set up button for heating time                        |
| ④ 감도조절버튼   | 4. set up button for sensitivity                         |

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2.Control the sensitivity level by sensitivity controlling button (4)

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The sensitivity may be different by the thickness of the wax pencil.  
As thicker, level needs to be increased. Find the adequate level of sensitivity to try several times. Sensitivity level operating shows as Green light in lamp (1) 조각도의 굵기에 따라 감도가 다를수 있습니다. (조각도의 굵기가 가늘수록 단계를 높이십시오. 구분표시램프가 녹색으로 바뀝니다.)

3. Set up the heating time of wax pencil pushing the time controlling button (3)

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As 1 level increase, heating time increases 0.4 seconds.

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From the kinds of wax pencil, the heating time may different.

Find and set up the proper heating time controlling the level.

Operating Heating time button shows as Red light in lamp (1).

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4. Operation

Put the wax pencil in the hole. Once the pencil is aware, LED light is on.

Then heating starts with bib sound as set up.

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( To gain better sensitivity, put the pencil in the center of hole. )

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## THE RANGE OF ENVIRONMENTAL CONDITIONS FOR THE PRODUCTS 사용환경

- \* Indoor use
- \* Temperature 5°C ~ 40°C \* Humidity Min. 50% ~ Max. 80%
- \* Main Supply voltage fluctuations up to  $\pm 10\%$  of the nominal voltage.
- \* POLLUTION DEGREE : II

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2 Years

### SERVICE FOR CHARGE

In case of the fault or malfunction from the misuse or improper use

- Using with wrong Input Volage (AC Voltage)
- Dropping down during setting or moving
- In case of user's misuse of improper use
- Malfunction after repairing at the places not recommended by manufacturer

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**SAESHIN PRECISION CO., LTD.**