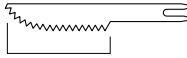
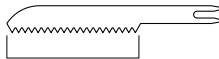
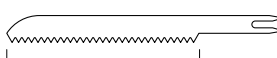
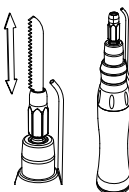
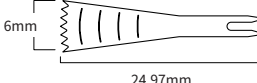
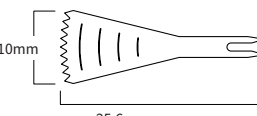
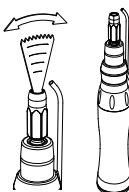
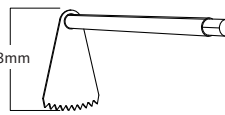
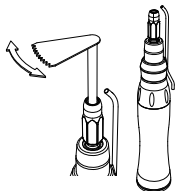
TRAUS Saw Blade

Please read this instruction manual carefully before an operation, and keep it for your future reference.

Saw Blades	Item	Reciprocating Saw Blades
	Picture	<div>① SAB-R15</div> <div></div> <div>15mm</div> <div>*Blade Thickness : 0.4mm</div> <div>② SAB-R20</div> <div></div> <div>20mm</div> <div>*Blade Thickness : 0.4mm</div> <div>③ SAB-R30</div> <div></div> <div>30mm</div> <div>*Blade Thickness : 0.4mm</div>
Applied Handpieces	Name	TRT12NRC
	Picture	

Saw Blades	Item	Sagittal Saw Blades
	Picture	<div>① SAB-S6</div> <div></div> <div>6mm</div> <div>24.97mm</div> <div>*Blade Thickness : 0.4mm</div> <div>② SAB-S10</div> <div></div> <div>10mm</div> <div>25.6mm</div> <div>*Blade Thickness : 0.4mm</div>
Applied Handpieces	Name	TRT12NSG
	Picture	

Saw Blades	Item	Oscillating Saw Blades
	Picture	<div>① SAB-O8</div> <div></div> <div>6.64mm</div> <div>*Blade Thickness : 0.4mm</div> <div>② SAB-O15</div> <div></div> <div>10.3mm</div> <div>*Blade Thickness : 0.4mm</div>
Applied Handpieces	Name	TRT12NOS
	Picture	

※ It's recommended to use the above products with SAESHIN's Micro Saw Handpiece

1. Intended Use

: Products to be mounted in the handpiece to puncture, perforate or cut the body tissue.

2. Components

: A blade, instruction manual

※ The applied handpieces are sold separately.

3. How to use

① This product is not sterilized when released. Make sure to sterilize the product before use. (Reference, article 5-2. Sterilization)

② Mount the blade in the handpiece and use it. (Reference, article 4. Blade Insertion & Removal)

③ After use, make sure to detach the blade, and clean and sterilize.

- Please check if the blade is fastened tightly after mounting it.
- Please do not connect or disconnect the blade until the handpiece has stopped completely.
- If you notice something wrong with the blade during use, stop using immediately and contact the manufacturer or the seller.
- Do not apply excessive force to the edge of the blade.

4. Blade Insertion & Removal

4-1) Insertion

① Turn the spanner counterclockwise to release the nose tip.

② Insert the blade. Push the blade in order to insert completely.

③ Turn the spanner clockwise to tighten the nose tip.

- Please check if the blade is fastened tightly before use.

4-2) Removal

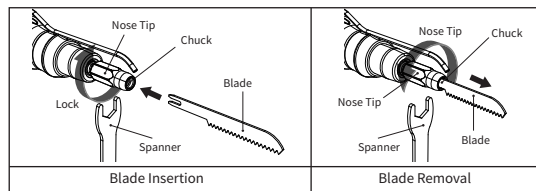
① Turn the spanner counterclockwise to release the nose tip.

② Remove the blade by pulling by hand.

③ After having removed the blade, turn the spanner clockwise, and tighten the nose tip lightly.

- Please do not tighten or loosen the blade excessively.

5. Maintenance

5-1) Cleaning

① Disconnect the blade to the handpiece.

② Wipe the blade with an alcohol swab.

- Please make sure to clean and autoclave the blade right after use before remaining blood has been dried.
- If remaining blood surface of the blade has coagulated, the blade may get rusty.

5-2) Sterilization

① Dry the product completely after cleaning.

② Put the product in a sterile bag and seal tightly.

③ Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132°C, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Dry Time	132°C
Moist heat sterilization	132°C	4 minutes	30 minutes	

- Make sure to use the sterilization bag and seal tightly.
- Make sure to dry the applicable products before sterilization.
- This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.

6. Warranty

SAESHIN guarantees this product for 1 year from invoice date. Product warranty means that SAESHIN has responsibility for defective material or operation failure. The product warranty does not cover user's misuse, wrong installation, inappropriate maintenance and repair, and normal wear of consumable parts such as ball bearing. In order for SAESHIN to confirm if the warranty is valid will require checking the operating condition, environment information, serial number and stamped invoice. The warranty will be performed as a repair or exchange according to manufacturer's analysis of product and judgement.

Warranty Exclusions

- Misuse and improper handling by users
- Dropping the products during installation, movement or use
- If the unit is damaged due to unspecified handpiece
- Normal wear
- Damage and failure due to natural disaster

[Precautions]

- Use of any other purpose is prohibited.
- The company did not sterilize the product. Please make sure to sterilize before operation. (Please refer to the 5.2 Sterilization)
- Do not drop the product or force with strong external shocks.
- Check for vibration other abnormal symptoms before use.
- If the product operates abnormally, stop immediately and contact the seller for check.
- Do not remove the blade while the handpiece is rotating.
- Make patient safety the top priority when use.
- To maintain it clean, always wipe the product with rubbing alcohol before use.
- Improper use may cause damage to the product. Be sure to use it by skilled technician or specialist.
- The manufacturer is not responsible for accidents caused by use not in accordance with the instruction manual.
- Product design and specifications are subject to change to improve quality.
- The user is responsible for the proper use and maintenance of the product.
- Do not dispose of this product with general waste.

Use conditions	Shipping/Storage condition
Temperature : 10~35 °C	Temperature : -20~60 °C
Humidity : 10~80%	Humidity : 10~90%
Atmospheric pressure : 700~1,060hPa	Atmospheric pressure : 500~1,060hPa

This product is the medical device.

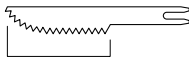
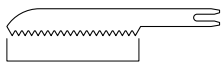
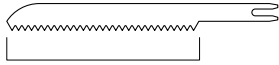
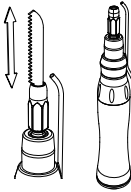
SAESHIN
SAESHIN PRECISION CO., LTD.
52, Secheon-ro 1-gil, Dasa-eup,
Dalseong-gun, Daegu, Korea
Tel. 82-53-587-2341 / Fax. 82-53-580-0999
www.saeshin.com / traus@saeshin.com

EC REP
MERIDIUS MEDICAL EUROPE LIMITED
Unit 3D, North Point House North point Business
Park New Mallow Road CORK, T23AT2P, Ireland
+353 212066448

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