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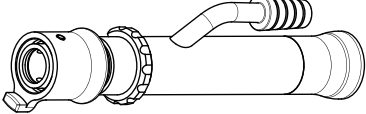

www.saeshin.com / traus@saeshin.com

SOM-S080P-KE [Rev.1] 2023-02-01

Before the operation, read the instruction manual carefully and keep it for future reference.

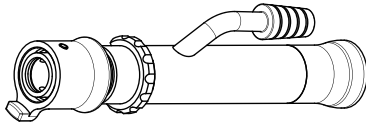Name	TRAUS TRT12NSH
Product Photography	
Weight	123g
Max. RPM	Max 6,000 rpm
Irrigation Type	External Irrigation Type
Blade replacement type	Chuck type
Coupling dimension	Standard dimension of dental handpiece (ISO 3964)

#### 1. Intended purpose

: For surgical bone cutting, shaving, polishing, grinding, drilling, endodontic treatment.

#### 2. Components


: Handpiece, Spray nozzle, Instruction manual

#### 3. Connection & disconnection of handpiece and motor

- Put the handpiece directly into the micro motor in order to connect them.
- Hold the handpiece and the micro motor, and then pull the handpiece from the micro motor in order to disconnect them.

- Please check if the connecting part is fastened tightly by pulling the handpiece from the micro motor smoothly.
- Please do not connect or disconnect the handpiece until the micro motor has stopped completely.
- Please use an E-type motor meeting ISO 3964 only.
- Please connect or disconnect the handpiece after mounting a blade.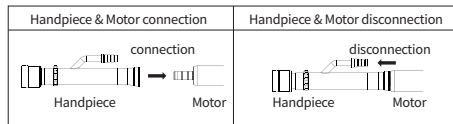

#### 4. Blade Insertion & Removal

##### 4.1 Insertion

- Before insertion, match the connecting grooves of the blade and the handpiece.
- Insert the blade after pulling the holder case down.
- Once the blade has been inserted to the end, push the holder case up and fix it.


- Please check if the blade is fastened tightly before use.

##### 4.2 Removal

- Pull the holder case down, and then remove the blade by pulling it.

- Please change the blade after the micro motor has stopped completely.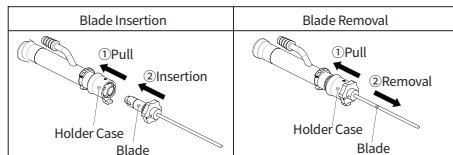

#### 5. Outer irrigation tube connection

- Outer irrigation nozzle is at the lower part of the handpiece.
- Pull the holder case down, and set the irrigation tube in the proper position to connect by turning the outer irrigation nozzle. Next, push the holder case up, and then fix the blade.
- Insert the irrigation tube into the end of the outer irrigation nozzle. Please check if the irrigation tube is connected tightly.

#### 6. How to connect the suction tube and how to use the suction valve

- Insert the suction tube into the suction pipe at the side of the handpiece. Please make sure to check if the tube is connected tightly.
- Use a suction tube of which internal diameter is 8.0~8.5mm.
- The suction starts working once the suction valve gets open by turning the suction valve clockwise. Turning counterclockwise, the valve gets locked and the suction stops working. It can be easily used to suit your needs.

#### 7. Maintenance of the product

##### 7-1) Cleaning


- Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
- Do not put tap water in the motor.
- Disconnect the handpiece to the motor.
- Remove contaminants on the handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- Rinse the handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- Wash the handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- Rinse the handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.

- Please make sure to clean, lubricate and autoclave the handpiece right after use before remaining blood inside the handpiece has been dried.
- If remaining blood inside the handpiece has coagulated, the handpiece may get rusty.

#### 7-2) Lubrication

- Insert the nozzle to hose connection part(the end part of angle) and spray lubricant 2~3 seconds.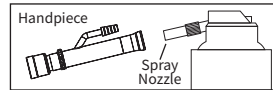

- Lubricate the product after each use.
- When lubrication, please hold the handpiece firmly because the handpiece can go out by the spray pressure.
- If the continuous operation, please put the oil every an hour.

#### 7-3) Sterilization

- Dry the applicable products completely after cleaning.
- Put the product in a sterile bag and seal tightly.
- Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132°C, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Dry Time	132°C
Moist heat sterilization	132°C	4 minutes	30 minutes	SSS

- This product is designed to withstand 250 times of re-sterilization process.
- Make sure to use the sterilization bag and seal tightly.
- Make sure to dry the applicable products before sterilization.
- Sterilizing the handpiece with blood or other remains may cause the breakage of the handpiece. Therefore, it is necessary to clean and lubricate the handpiece before sterilization.
- This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.

#### 8. WARRANTY

SAESHIN guarantees out product for 1 year from invoice date, and product warranty means that SAESHIN has responsibility of defective material or operation failure. The product warranty does not cover user's misuse, wrong installation, inappropriate maintenance and repair, and normal wear of consumables such as bearings. To check the warranty requires the operating condition, environment information, serial number of the product and stamped invoice. The warranty will be performed as repairs or exchanges by manufacturer's judgement and analysis of product.

#### Exception of warranty

- Misuse and improper handling by users
- Dropping the products during installation, movement or use
- If the unit is damaged due to unspecified consumable parts or accessories
- If the product is not repaired by Saeshin or the repair shop designated by Saeshin.
- Normal wear of consumable parts (Ball Bearing or etc.)
- Damage and failure due to natural disaster
- If the product is disassembled or modified at will.

#### [Caution for use]

- Use of any other purpose is prohibited.
- The company did not sterilize the product. Please make sure to sterilize before operation. (Please refer to the 7.3 Sterilization)
- Do not drop the product or force with strong external shocks.
- Do not disassemble or modify the product.
- Check for vibration, heat, noise and other abnormal symptoms before use.
- If the product operates abnormally, stop immediately and contact the seller for check.
- Do not remove the motor while the product is rotating.
- Moisture and impurities may cause malfunction of the product. Please clean and dry with air.
- Make patient safety the top priority when use.
- To maintain it clean, always wipe the product with rubbing alcohol before use.
- Improper use may cause damage to the product. Be sure to use it by a skilled technician or specialist.
- The manufacturer is not responsible for accidents caused by use not in accordance with the instruction manual.
- The user is responsible for the proper use and maintenance of the product.
- This product should not be disposed with general waste.

Use conditions	Shipping/Storage conditions
Temperature: 10~35°C	Temperature: -20~60°C
Humidity: 10~80%	Humidity: 10~90%
Atmospheric pressure: 700~1,060hPa	Atmospheric pressure: 500~1,060hPa

This product is the medical device.

**SAESHIN**  
SAESHIN PRECISION CO., LTD.  
52, Secheon-ro 1-gil, Dasa-eup,  
Dalseong-gun, Daegu, Korea  
Tel. 82-53-587-2341 / Fax. 82-53-580-0999  
www.saeshin.com / traus@saeshin.com

CE 1639  
MERIDIUM MEDICAL EUROPE LIMITED  
Unit 3D, North Point House North point  
Business Park New Mallow Road CORK,  
T25AT 2P, Ireland  
+353 21 2066448  
SOM-S080P-KE [Rev.1] 2023-02-01