

Before the operation, read the instruction manual carefully and keep it for future reference.

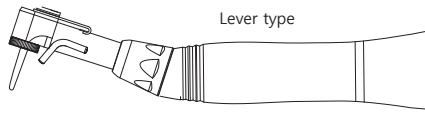Lever type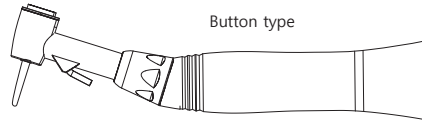

Button type

Name	ACL-41I	ACL-42I	ACL-43I	ACL-45I	ACL(B)-41I	ACL(B)-42I	ACL(B)-43I	ACL(B)-45I	ACL(B)-61I	ACL(B)-62I	ACL(B)-63I	ACL(B)-65I
Max. speed (rpm)	2,500	3,100	780	1,560	2,500	3,100	780	1,560	2,500	3,100	780	1,560
Gear ratio	20:1	16:1	64:1	32:1	20:1	16:1	64:1	32:1	20:1	16:1	64:1	32:1
Weight	60g	60g	65g	60g	61g	61g	65g	61g	48g	48g	53g	48g
Bur replacement type	Lever type				Button type							
Articles	Low speed angle											
Chuck type	Type1. by ISO 1797											
Bur dimension	CA bur (Ø 2.35)											
Length of Shank	9~12mm											
Max. overall length of rotary instrument	30mm											
Coupling dimension	Standard dimension of dental handpiece (ISO 3964)											

- The maximum speed is different with a output of the control box.

1. Intended use

Contra angle handpieces are intended for a wide range of dental procedures including:

A. Implant placement, including

- 1) Preparation of the osteotomy site
- 2) Bone contouring, osteoplasty

B. Periodontal surgeries

- 1) Bone contouring & alveoplasty around living teeth
- 2) Removal of exostosis

C. Bone grafting

- 1) Preparation of the donor site (for e.g. symphysis and ascending rames etc.)
- 2) Harvesting autogen living bone
- 3) Sinus elevation & grafting of alveolar sockets

D. Removal and sectioning of teeth and teeth bone for e.g. impacted third molars and complicated extractions

2. Components

Angle, Spanner

3. Bur replacement

- Lever type: Turn the lever counterclockwise to replace the bur.
- Button type: After press the button and remove the bur, replace the bur.

※ Please use ISO standard bur and avoid to use of rusty or curved bur because it can cause excessive wear of the bearing.
※ Please comply with instructions of bur. (Safety speed and precautions)



4. Maintenance of the product

1) Cleaning

- ① Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
- ② Disconnect the angle handpiece to the motor.
- ③ Remove contaminants on the angle handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- ④ Rinse the angle handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- ⑤ Wash the angle handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- ⑥ Rinse the angle handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- ⑦ Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.


2) Lubrication

- ① Disassemble the product into 3 parts.
 - A. Fixed a wrench to the joint cap and turn joint cap counterclockwise to open.
 - B. Take out the gear.
- ② See the picture below, put the nozzle with the right direction and spray lubricant for 2~3 seconds.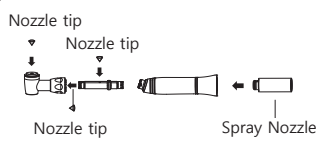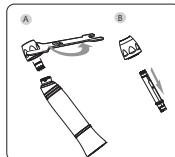

- ③ Reassemble the product.
 - A. Insert the gear into the head assembly.
 - B. Make sure the gear fits into the head assembly and check it turning smoothly.
 - C. Assemble the head part and body as right direction.
 - D. Use a wrench to lock the joint cap tightly.

- ④ After reassembly, insert the nozzle to handle and spray lubricant 2~3sec. See the picture below.

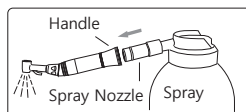- The nozzle tip, spray nozzle are not provided by manufacturer.
- After the lubrication, please put the gear right direction.
- If put the gear wrong direction, the handpiece can be damaged.
- If the continuous operation, please put the oil every an hour.
- When lubrication, please hold the handpiece firmly because the handpiece can go out by the spray pressure.

3) Sterilization

- ① Put the angle in a sterile bag and seal tightly.
- ② Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132°C, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Dry
Moist heat	132°C	4minutes	30minutes

- ※ This product is designed to withstand 250 times of re-sterilization process.
- ※ This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.
- ※ The product is intended to be wrapped in an FDA cleared sterilization pouch/bag prior to sterilization.

! [Caution for use]

- This product shall be used for intended purpose only.
- This product is not in sterilization once release by manufacturer. Please certainly do user sterilization before using. (Refer to 'Maintenance of product - sterilization')
- If sterilize the product with rest of blood, the product can be damaged. So, please fully clean up and lubricate the product.
- Do not use this product for wrong purpose and be aware of the safety.
- Keep the recommended loading & resting time by manufacturer of control box. If not, patient might get burn of damage by overheating.
- After stopping the motor perfectly, disassemble the product from the motor.
- Disassemble or repair the product only the case which is mentioned in instruction manual.
- Do not drop the product. It can be damaged.
- Please insert the product to the end of E-type motor shaft and use it.
- Verify the product's operation, especially vibration, noise, and temperature rise are normal.
- Please contact the dealer/store or manufacturer when problem occurs.
- The manufacturer does not have any responsibility for any accidents that user did not follow the instruction manual.
- User has responsibility for the use and maintenance of the product.
- US federal law restricts this device to sale by or on the order of dentist.
- If you want to get an instruction manual in other languages, please contact the manufacturer.

Operating condition	Transport and storage condition
Temperature : 10~35°C	Temperature : -20~60°C
Humidity : 10~80%	Humidity : 10~90%
Atmospheric pressure : 700~1,060hPa	Atmospheric pressure : 500~1,060hPa

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MERIDUS MEDICAL EUROPE LIMITED
Unit 30, North Point House North point Business Park New Mallow Road CORK,
T23AT29, Ireland
SOM-S019P-KE [Rev11] 2023-02-01

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