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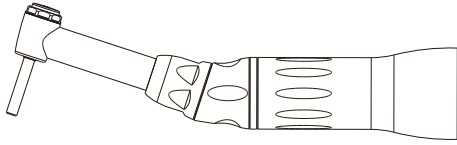
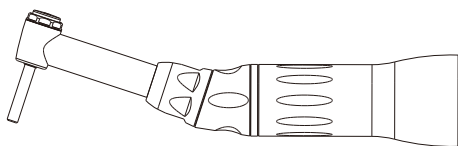
Before the operation, read the instruction manual carefully and keep it for future reference.

Name	ACL(B)-42EM
Weight	52g
Gear ratio	16:1
Bur replacement type	Button type
Bur dimension	CA bur (Ø 2.35)
Coupling dimension	Standard dimension of dental handpiece (ISO 3964)
Chuck type	Type1. by ISO 1797

### 1. Intended purpose

The handpiece used in root canal treatment.

### 2. Components

Angle, Wrench, Spray nozzle, Instruction manual

### 3. Bur replacement

- Button type: After press the button and remove the bur, replace the bur.

- ※ Please use ISO standard bur and avoid to use of rusty or curved bur because it can cause excessive wear of the bearing.
- ※ Please comply with instructions of bur. (Safety speed and precautions)

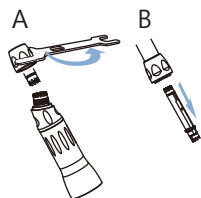
### 4. Maintenance of the product

#### 1) Cleaning


- ① Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
- ② Disconnect the angle handpiece to the motor.
- ③ Remove contaminants on the angle handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- ④ Rinse the angle handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- ⑤ Wash the angle handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- ⑥ Rinse the angle handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- ⑦ Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.

#### 2) Lubrication

- ① Disassemble the product into 3 parts.
  - A. Fixed a wrench to the joint cap and turn joint cap counterclockwise to open.
  - B. Take out the gear.

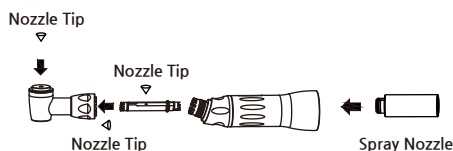- ② See the picture below, put the nozzle with the right direction and spray lubricant for 2~3 seconds.

#### ③ Reassemble the product.

- A. Insert the gear into the head assembly.
- B. Make sure the gear fits into the head assembly and check it turning smoothly.
- C. Assemble the head part and body as right direction.
- D. Use a wrench to lock the joint cap tightly.

- The nozzle tip is not provided by manufacturer.
- After the lubrication, please put the gear right direction. If put the gear wrong direction, the handpiece can be damaged.
- If the continuous operation, please put the oil every an hour.
- When lubrication, please hold the handpiece firmly because the handpiece can go out by the spray pressure.

#### 3) Sterilization

- ① Put the angle in a sterile bag and seal tightly.
- ② Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132°C, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Dry
Moist heat sterilization	132°C	4minutes	30minutes

- ※ This product is designed to withstand 250 times of re-sterilization process.
- ※ This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.

### ! [Caution for use]

- This product shall be used for intended purpose only.
- This product is not in sterilization once release by manufacturer. Please certainly do user sterilization before using. (Refer to #5 sterilization)
- If sterilize the product with rest of blood, the product can be damaged. So, please fully clean up and lubricate the product.
- Do not use this product for wrong purpose and be aware of the safety.
- After stopping the motor perfectly, disassemble the product from the motor.
- Disassemble or repair the product only the case which is mentioned in instruction manual.
- Do not drop the product. It can be damaged.
- Please insert the product to the end of E-type motor shaft and use it.
- Verify the product's operation, especially vibration, noise, and temperature rise are normal.
- Please contact the dealer/store or manufacturer when problem occurs.
- The manufacturer does not have any responsibility for any accidents that user did not follow the instruction manual.
- Design and specification of product can be changed to improve quality of product.
- User has responsibility for the use and maintenance of the product.
- US federal law restricts this device to sale by or on the order of dentist.
- If you want to get an instruction manual in other languages, please contact the manufacturer.

Operating condition	Transport and storage condition
Temperature : 10~35°C	Temperature : -20~60°C
Humidity : 10~80%	Humidity : 10~90%
Atmospheric pressure : 700~1,060hPa	Atmospheric pressure : 500~1,060hPa

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EC REP

**MERIDIUS MEDICAL EUROPE LIMITED**  
Unit 3D North Point House North point  
Business Park New Mallow Road CORK,  
T23A12P, Ireland  
+353 212066448

SOM-S033P-KE (Rev.3) 2020-03-01

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SAESHIN PRECISION CO., LTD.