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
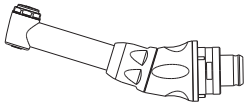
N STRONG CONTRA ANGLE HANDPIECE

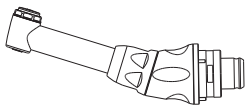Before the operation, read the instruction manual carefully and keep it for future reference.

Name	ACL(B)-41EP	ACL(B)-42EP	ACL(B)-45EP	ACL(B)-46EP
Weight	34.8g			
Gear ratio	20:1	16:1	32:1	10:1
Bur replacement type	Button type			
Bur dimension	CA bur (Ø 2.35)			
Chuck type	Type1. by ISO 1797			

1. Intended purpose

The handpiece used in root canal treatment.

2. Components

Angle, Wrench, Instruction manual

3. Bur replacement

- Button type : After press the button and remove the bur, replace the bur.

※ Please use ISO standard bur and avoid to use of rusty or curved bur because it can cause excessive wear of the bearing.

※ Please comply with instructions of bur. (Safety speed and precautions)

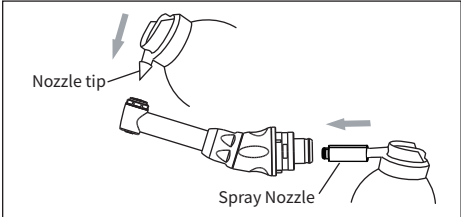
4. Maintenance of the product

1) Cleaning

- ① Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
- ② Disconnect the angle handpiece to the motor.
- ③ Remove contaminants on the angle handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- ④ Rinse the angle handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- ⑤ Wash the angle handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- ⑥ Rinse the angle handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- ⑦ Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.

2) Lubrication

- ① Insert the spray nozzle to motor assembly part (the end part of angle) and spray lubricant 2~3 seconds.
- ② Insert the nozzle tip to head part and spray lubricant 2~3 seconds.

- The nozzle tip and spray nozzle are not provided by manufacturer.
- If the continuous operation, please put the oil every an hour.
- When lubrication, please hold the handpiece firmly because the handpiece can go out by the spray pressure.

3) Sterilization

- ① Put the angle in a sterile bag and seal tightly.
- ② Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132°C, and let it dry for 30 minutes.

Sterilization	Temperature	Time	Dry
Moist heat sterilization	132°C	4minutes	30minutes

※ This product is designed to withstand 250 times of re-sterilization process.

※ This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.

[Caution for use]

- This product shall be used for intended purpose only.
- This product is not in sterilization once release by manufacturer. Please certainly do user sterilization before using. (Refer to 'Maintenance of product – sterilization')
- If sterilize the product with rest of blood, the product can be damaged. So, please fully clean up and lubricate the product.
- Do not use this product for wrong purpose and be aware of the safety.
- After stopping the motor perfectly, disassemble the product from the motor.
- Disassemble or repair the product only the case which is mentioned in instruction manual.
- Do not drop the product. It can be damaged.
- Please insert the product to the end of control unit and use it.
- Verify the product's operation, especially vibration, noise, and temperature rise are normal.
- Please contact the dealer/store or manufacturer when problem occurs.
- The manufacturer does not have any responsibility for any accidents that user did not follow the instruction manual.
- Design and specification of product can be changed to improve quality of product.
- User has responsibility for the use and maintenance of the product.
- US federal law restricts this device to sale by or on the order of dentist.
- If you want to get an instruction manual in other languages, please contact the manufacturer.

Operating condition	Transport and storage condition
Temperature : 10~35°C	Temperature : -20~60°C
Humidity : 10~80%	Humidity : 10~90%
Atmospheric pressure : 700~1,060hPa	Atmospheric pressure : 500~1,060hPa

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EC REP

**MERIDIUS MEDICAL EUROPE LIMITED**

Unit 3D, North Point House North point  
Business Park New Mallow Road CORK,  
T23AT2P, Ireland  
+353 212066448

SOM-S072P-KE Rev.7 2023-06-29

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**SAESHIN PRECISION CO., LTD.**