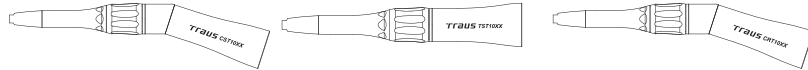
Traus

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
SOM-S063P-KE [Rev.1] 2023-03-23

Traus**CST / TST / CRT Angle Handpiece**

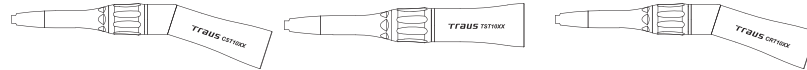Please read this Operation Manual carefully before use, and file for future reference

| Model Name | CST10XX | TST10XX | CRT14XX |
|----------------------|---|---------|---------|
| Weight | 124g | 89g | 123g |
| Gear Ratio | 1:1 | | |
| Bur Replacement Type | Chuck Open type | | |
| Bur Dimension | Cylindrical bur (Ø 2.35) | | |
| Coupling dimension | Standard dimension of dental handpiece (ISO 3964) | | |

1. Intended purpose

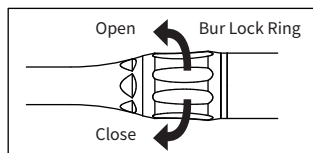
: For surgical bone cutting, shaving, polishing, grinding, drilling, endodontic treatment.

2. Components

: Angle, Wrench, Spray nozzle, Instruction manual

3. Bur replacement

Turn the handle clockwise to open chuck. After open the chuck, replace the bur.


- ※ Please use ISO standard bur and avoid to use of rusty or curved bur because it can cause excessive wear of the bearing.
- ※ Please comply with instructions of bur. (Safety speed and precautions)

4. Maintenance of the product**1) Cleaning**

- ① Rotate the head in a tap water about 10 seconds with maximum speed to remove the remains (blood, physiologic saline solution, etc.) and rinse.
- ② Disconnect the angle handpiece to the motor.
- ③ Remove contaminants on the angle handpiece using a nylon brush after soaking for 20 minutes in Enzymatic Detergent.
- ④ Rinse the angle handpiece using the tap water for 3 minutes to remove Enzymatic Detergent.
- ⑤ Wash the angle handpiece for 10 minutes in the ultrasonic cleaner containing Enzymatic Detergent.
- ⑥ Rinse the angle handpiece for 3 minutes in the ultrasonic cleaner containing purified water repeatedly subjected 3 times. (Change the purified water each time.)
- ⑦ Wipe using dust-absorbent after the product soaked in alcohol(70%) for 30 seconds.

2) Lubrication

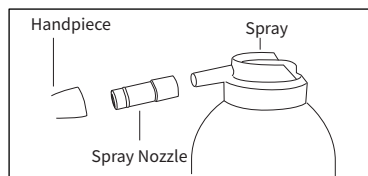See the picture below, insert the lubricant spray nozzle to the spray. And, insert the nozzle to the end part of angle (motor assembly part) and spray lubricant 2~3 seconds.

- ⚠ - If the continuous operation, please put the oil every 1hour.
- ⚠ - When lubrication, please hold the handpiece firmly because the handpiece can go out by the spray pressure.

3) Sterilization

- ① Put the angle in a sterile bag and seal tightly.
- ② Put the sterile bag in the sterilizer and sterilization for 4 minutes with 132°C, and let it dry for 30 minutes.

| Sterilization | Temperature | Time | Dry |
|--------------------------|-------------|-----------|------------|
| Moist heat sterilization | 132°C | 4 minutes | 30 minutes |

※ This product is designed to withstand 250 times of re-sterilization process.

※ This product recommends the moist heat sterilization (autoclave), other sterile methods are not guaranteed the effectiveness.

! [Caution for use]

- This product shall be used for intended purpose only.
- This product is not in sterilization once release by manufacturer. Please certainly do user sterilization before using. (Refer to 'Maintenance of product - sterilization')
- If sterilize the product with rest of blood, the product can be damaged. So, please fully clean up and lubricate the product.
- Do not use this product for wrong purpose and be aware of the safety.
- After stopping the motor perfectly, disassemble the product from the motor.
- Disassemble or repair the product only the case which is mentioned in instruction manual.
- Do not drop the product. It can be damaged.
- Please insert the product to the end of E-type motor shaft and use it.
- Verify the product's operation, especially vibration, noise, and temperature rise are normal.
- Please contact the dealer/store or manufacturer when problem occurs.
- The manufacturer does not have any responsibility for any accidents that user did not follow the instruction manual.
- User has responsibility for the use and maintenance of the product.
- US federal law restricts this device to sale by or on the order of dentist.
- If you want to get an instruction manual in other languages, please contact the manufacturer.

| | Use conditions | Shipping/Storage conditions |
|----------------------------|----------------|-----------------------------|
| Temperature (°C) | 10~35 | -20~60 |
| Humidity (%) | 10~80 | 10~90 |
| Atmospheric pressure (hPa) | 700~1,060 | 500~1,060 |

This product is the medical device.

**SAESHIN**
SAESHIN PRECISION CO., LTD.

52, Secheon-ro 1-gil, Dasa-eup,
Dalseong-gun, Daegu, Korea
Tel. 82-53-587-2341 / Fax. 82-53-580-0999

www.saeshin.com/traus@saeshin.com

EC REP
MERIDIUS MEDICAL EUROPE LIMITED
Unit 3D, North Point House North point
Business Park New Mallow Road CORK,
T23AT2P, Ireland
+353 21 2066448

SOM-S063P-KE [Rev.1] 2023-03-23