INSTRUCTION MANUAL
wp-1 / wp-3

Thank you for your purchase.

In order to have trouble-free performance for many years, please read all instructions before operating.
Please keep this manual for future reference.

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[DESCRIPTION / 용도]

This product is the electric equipment to melt the rigid wax using in dental laboratory. By the minimized temperature deviation and exact temperature controlling, the wax is melt without any changes in physical properties and the wax condition helps to perform the precise and elaborate works.

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[Model / 모델]

wp-1 : wax dipping unit with one pot

wp-3 : wax heating unit with three pot to set the temperature of each pot
wax dipping is also possible from the enough depth of pots.

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Connect the power cord from the wax dipping unit and the electricity.
Switch on the On/Off power switch located back side.

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By ▲/▼ button, set the temperature.

(30°C-130°C / 86°F-266°F)

Keep pressing down the button continuously after 2-3 times. The temperature reaches rapidly.
The set temperature is shown about 2 seconds on the display window and then the current temperature is shown.

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wp-3

Choose the pot to use by pushing Mode button.

The display window shows "OFF".

Press down button and set the temperature. (30°C-130°C / 86°F-266°F)

Keep pressing down the button continuously after 2-3 times. The temperature reaches rapidly.

The set temperature is show about 2 seconds on the display window and then the current temperature is shown.

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3. Display Centigrade & Fahrenheit / 화씨(°F) 섭씨(°C) 변환

Push °C/°F button for 2 seconds. Temperature system changes automatically.

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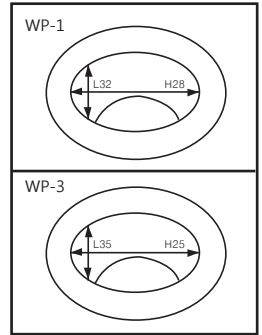
wp-1

The heat-led (see the picture) blinks to reach the required temperature and also blinks when the temperature reaches +5°C to the set temperature. Reaching to the set temperature, the heat-led turns off.

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wp-3

wp-3

If the heater is operating to reach to the set temperature, the led on the top of the display window shows red and turns to green when the temperature reaches.

-ON/OFF of heater-

The heater is operating when the temperature is shown by press ▲button. Pressing ▼ button, the temperature is decreasing by 30℃ / 86°F and the display window shows "OFF". The heater turns off.

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5. Beep Sound / 부저

Beep sound hear on pressing each button for ensuring input.

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[CAUTIONS / 주의사항]

- Do not touch the dipping unit while in use as it's very hot.- Turn off the power in case of not using the wax dipping unit,
- Do not touch the dipping unit with wet hands.
- The accurate circuit board is included in the dipping unit so that it is necessary to avoid any impact or force which may cause mechanical troubles.
- Contact the manufacturer or distributors for questions or more information regarding the trouble-shooting.

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2 Years

SERVICE FOR CHARGE

In case of the fault or malfunction from the misuse or improper use

- Using with wrong Input Volage (AC Voltage)
- Dropping down during setting or moving
- In case of user's misuse of improper use
- Malfunction after repairing at the places not recommended by manufacturer

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[For your safety, Keep to the following / 안전을 위해 꼭 지켜주세요.]

= Do not use with damaged or faulty power cord, plug, and outlet. Contact to the manufacture.

It may cause the danger of fire, electric shock.

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= When unplug the power cord, hold the plug and do not touch with wet hand.

Do not use= Do not bend the power cord excessively and do not press with heavy things.

It may cause electric shock and fire.

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= Use the grounding conductor. Breakage or electric leak may cause the danger of electric shock.

For grounding, inquiry to electrician or to main office.

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= Do not place near heat equipment and candles or cigarette causing fire.

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= Close supervision needs if this product is used by or near by the very young, the elderly or invalid.

Do not leave children unattended in any room where this product has been installed.

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= In case of using extension cord, do not use many electricity equipments in one time.

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= Do not place or set up the product in the humid and rainy or snowy location.

It brings the danger of electric shock and fire.

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= In case of malfunction, dropping, drowning, do not use and contact to the manufacturer

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Saeshin is not responsible for the accident causing by not following the instructions,

Design and specification of the product can be subject to be changed for improving the quality.

informing the product, model, faulty status and the customer's contact information (tel # or email etc),
the kind consulting and quick service will be provided.

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SAESHIN**SAESHIN PRECISION CO., LTD.**